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**GEOTEK® SUPRAPUBİK DRAINAGE CATHETER****Instruction for Use****ENG**

It is a complete set which includes; Silicone coated Folley catheter, Peel away trocar and scalpel.

**Indications:** is intended use for pre or post operative bladder drainage via suprapubic way

**Contraindications:** Serious infections, hematuria, incontinence

**Side Effects:** Kidney, bladder irritation caused by foreignbody, pain, infection, increased miction-frequency, hematuria

**Cautions:** This Kit should be used by only by physicians experienced in the use of such device.

Regular monitoring of location and free passage of catheter is necessary to evaluate darinage efficiency.

Ethylene oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Do not use over expired products. Store in dry place, between 5 – 30 °C temperature, protect from sun light. Possible allergic reactions should be considered. For single Patient Use Only. Do not attempt to clean or resterilize this product. Disposal as a biohazard after use.

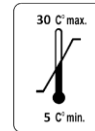
**Placement of Suprapubic Catheter:**

- Apply local anesthetic to before introducing the catheter and confirm catheter location with aspirating urine.
- Make small incision to the skin
- Insert peel away sheath and advance it carefully.
- Remove the trocar and insert the folley catheter through sheath.
- Inflate the balloon of folley catheter and fix the catheter in to bladder wall.
- Fix the catheter to skin with fixation disk.
- Remove the sheath by tearing down along the folley catheter.

**Note:** These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

**CE**

1984: CE Mark and NB identification number

Single Use  
sterilized

Storage limits

Expiry Date

STERILE EO

Ethylene oxide

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**GEOTEK MEDİKAL VE SAĞLIK HİZMETLERİ LTD.ŞTİ**